

PacSeal™

PIT & FISSURE SEALANT



PacSeal™ is an opaque, resin-based, and light cured pit and fissure sealant. PacSeal™ is specially designed with wet-bonding resin technology that ensures an ideal, microleakage-free marginal seal without any teeth isolation. Its chipping and cracking free formula is easily delivered from pre-filled syringes with 20 matching brush tips.

PRODUCT FEATURES

- + Margin-free
- + No drying or bonding agents required
- + Extended fluoride releasing formula
- + Quick and easy delivery system



BUY 3 GET 1 FREE!*

Net.Price
\$34.60
(Reg..\$46.13)

ITEM#	DESCRIPTION
-------	-------------

PD-138	4 x 1.2cc PacSeal syringes, Natural shade 20 X brush tips
--------	---

Pac-Dent®
Passion for Excellence

Directions

1. Thoroughly clean enamel surfaces with oil-free, fluoride cleaning paste.
2. Rinse well, removing all residual paste from pits and fissures.
3. Prepare targeted enamel in the usual manner.
4. Isolate treatment area and dry lightly with oil-free compressed air for 15 seconds.
5. Apply 38% phosphoric acid etching gel to the clean tooth surface for 60 seconds. Rinse etched area for 15 seconds and dry thoroughly. If area does not appear chalky and frosty, etch again.
6. Place applicator tip securely on PacSeal syringe and carefully start flow of sealant. Use a new applicator tip for each patient. Recap syringe after use.
7. Apply PacSeal to the prepared occlusal surface. For pit and fissure sealing, flow the material from cusp to cusp, but do not cover the marginal ridges and avoid air bubble entrapment. Two drops of sealant material are sufficient to treat the occlusal surface on one quadrant.
8. Light cure with iCure LED curing light, expose all coated surface for 30-40 seconds, keeping the light 1-2mm from the surface.
9. After sealant has set, wipe surface with cotton roll. Check for hardness and retention.
10. Store in a cool, dry place away from heat and sunlight.



BUY 3 GET 1 FREE!*

Net.Price
\$34.60
(Reg..\$46.13)

ITEM#	DESCRIPTION
-------	-------------

PD-138	4 x 1.2cc PacSeal syringes, Natural shade 20 X brush tips
--------	---



Manufactured for Pac-Dent, Inc.
670 Endeavor Circle, Brea, CA 92821
www.pac-dent.com · info@pac-dent.com **Made in USA**

*To receive your free goods, submit a copy of your dealer invoice to freegoods@pac-dent.com. Free goods will be shipped directly from Pac-Dent. Allow 4-6 weeks for delivery. Cannot be combined with any other offer. No cash value.