


iVoc™ LED Piezo Ultrasonic Scaler

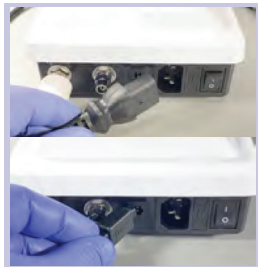

Installing cords:

Insert power cord plug into socket. Insert foot pedal cord plug to socket.

iVac™ LED Piezo Ultrasonic Scaler voltage is: 110V




If you are going to use water line:

Connect one end of the water pipe (1/4") to the water entrance, and the other end to clean water source.



Use handpiece and cable connection alignment guides to install handpiece.

Ensure there is no gap.

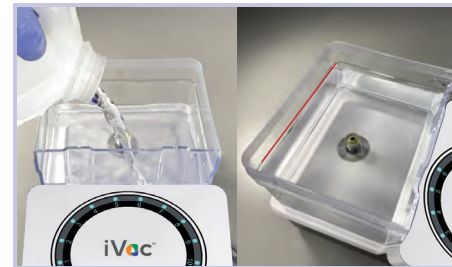

If you are going to use the reservoir (tank):

Fill your reservoir up to the fill line with distilled water or your preferred irrigant solution.



Turn irrigation flow control knob to maximum by turning the knob counterclockwise.



Turn on the power switch



1-10 - Power (amplitude)

G, P and E - Vibration modes

Double E - Endo irrigation mode
water line Reservoir / bottle



Use touch panel for adjusting water/irrigant supply mode (water line or reservoir), and vibration intensity modes (G, P, or E)

Vibration Intensity Modes:


G - Scaling, Hygiene

P - Periodontics, and Cavity Preparation

E - Endodontics

Double E - Irrigation


iVac™ LED Piezo Ultrasonic Scaler

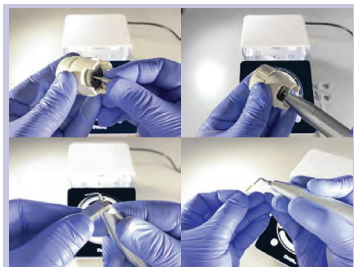

Insert the desired tip:

Use the torque wrench for the scaling and perio tips.

Use wrench for endo tips.

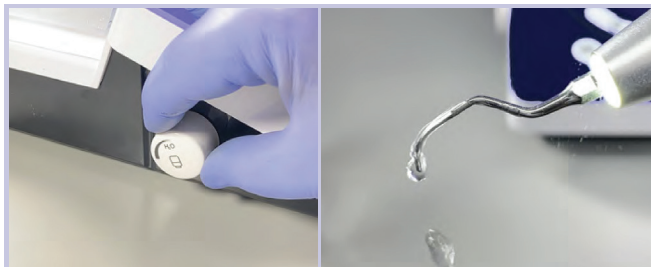

Water / irrigant volume adjustment:

Step on foot switch. Tip begins to vibrate.

Turn irrigation flow control knob clockwise/counterclockwise to adjust.



Operating power adjustment (scale 1 to 10)

Start the power at minimal rate (1), adjusting/increasing during clinical treatment according to the procedure development.

i IMPORTANT:



Cleaning mode (purge cycle):

Fill reservoir with distilled water to a quarter of its capacity. Reinstall reservoir. Start by pressing **simultaneously** the 2 buttons on the touch screen panel (water line and reservoir buttons). A single beep alarm will sound and the reservoir button will start to blink. Step on the foot switch once to start cycle. A continuous flow of distilled water will clean the system internally. After cleaning cycle, press the 2 buttons simultaneously again to exit cleaning mode.

iVac S 1-7 (E)

ED12D	1-6 (E)
ED17	1-3 (E)
ED15	1-3 (E)
ED 70 (with Polysonic tip)	1-3 (E)

Endo Tips Power Table



Scan QR Code for more information.